

microdot[®]+

Cleaning Your Meter

1. First remove the anti-microbial cover. Use a lint free cloth dampened with soapy water or alcohol (70%-80%) to clean the outside of the blood glucose meter.
2. Microdot+ Meter Disinfecting Guideline: To disinfect the meter, dilute 1ml of household bleach in 9ml of water to achieve a 1:10 dilution. The solution can then be used to dampen a paper towel **Do Not Saturate The Towel**. Then use the dampened towel to thoroughly wipe down the meter. Please note that there are commercially available 1:10 bleach wipes from a variety of manufacturers.
3. Germicidal wipes are readily available through major medical distributors. To use the products, remove a wipe from the container and follow product label instructions to disinfect the meter.
4. Always take extreme care not to get liquid in the test strip port of the meter.

Microdot+ Customer Care

Open Monday – Friday 9am-5pm

Telephone:

01480 482920

Address:

Cambridge Sensors Limited
Units 9 & 10 Cardinal Park
Godmanchester
Huntingdon
Cambs
PE29 2XG

Email:

reception@cs-limited.co.uk